



Customized Pelvic Floor Muscle Trainer Tens Therapy Incontinence Stimulator device

Our Product Introduction

Basic Information

- Place of Origin: China
- Brand Name: JaneJoy
- Model Number: J-02
- Minimum Order Quantity: 100
- Price: 19USD one set
- Packaging Details: one set one color box , 50sets one carton
- Delivery Time: 25days lead time
- Payment Terms: , T/T
- Supply Ability: 10000pcs per month



Product Specification

- Material: ABS
- Warranty: 1year
- After-sale Service: Return And Replacement
- Instrument Classification: Class II
- APPLICATION: Pelvic Floor Trainer For Women Postpartum
- Packing: 1 Set/ Cloth Bag
- Function: Muscle Trainer For Pelvic Floor Muscle Repair
- MOQ: 100sets
- Service: ODM OEM Service
- Color: White And Pink
- Highlight: **Class II Tens Pelvic Floor Device,
Customized Logo Pelvic Floor Muscle
Stimulator
, Customized Pelvic Floor Muscle Trainer**

Product Description

Tens Therapy Incontinence Stimulator Exercisor Internal Pelvic Floor Muscle Trainer For Women Postpartum

Key attributes

Industry-specific attributes

| | |
|--------------------|------------------------|
| Warranty | 1 Year |
| After-sale Service | Return and Replacement |

Other attributes

| | |
|---------------------------|---|
| Place of Origin | Guangdong, China |
| Instrument classification | Class II |
| Brand Name | BYTECH/OEM |
| Model Number | VT10 |
| APPLICATION | pelvic floor trainer for women postpartum |
| CERTIFICATION | CE |
| Product name | Pelvic floor trainer |
| Packing | 1 set/ cloth bag |
| Function | muscle trainer for pelvic floor muscle repair |
| MOQ | 50 Sets |
| Service | ODM OEM Service |
| Color | White/Customized Color |
| Logo | Customized Logo Availale |
| Advantage | Competitive |

Packaging and delivery

| | |
|-------------------|--|
| Packaging Details | 1SET/ cloth bag, custom design welcome |
| Port | Shenzhen or Guangzhou |

Supply Ability


| | |
|----------------|------------------------------|
| Supply Ability | 10000 Piece/Pieces per Month |
|----------------|------------------------------|

[Show less](#)


Lead time

| | | |
|------------------|--------|------------------|
| Quantity (sets) | 1 - 50 | > 50 |
| Lead time (days) | 15 | To be negotiated |


Customization

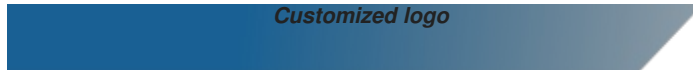

Min. order: 10000

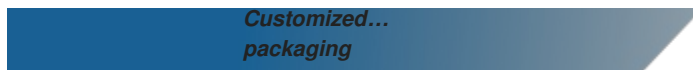

Min. order: 10000

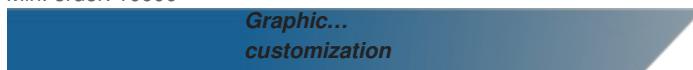

Min. order: 10000

For more customization details, [message supplier](#)

Product descriptions from the supplier

Overview

Video Description

Product Description

What is Pelvic Floor Stimulator(PFS)

It is an ultra simple, yet powerful digital Pelvic Floor Stimulator (PFS). Compact, stylish and designed for use by women of all ages to help strengthen pelvic floor muscles.

PFS can help women gain greater bladder control, increased vaginal tone and greater sensation during intercourse.

One in three women suffer from incontinence during their lives. It can occur at any time, most commonly after childbirth. Pelvic Floor Stimulation strengthens the pelvic floor by passing mild electrical pulses via an internal vaginal probe to contract and relax the pelvic floor muscles.

| | |
|--|---|
| Name | Vaginal Probe with Indicator |
| Output Channel | Single |
| Program | Urge ,Stress ,Mixed ,After Care |
| Output Voltage | 0-40V adjustable in 15 steps across a 500 Ω load |
| Output Intensity | 0-80mA adjustable in 15 steps across a 500 Ω load |
| Output Waveform | Biphasic rectangular |
| Frequency | Refer to Point 6 for treatment program |
| Safety | Automatic 5 minutes power Off |
| Battery | 2 x 1.5v AAA Batteries |
| Battery low detect | 2.2V \pm 0.1V |
| Automatic Power Off | If the output remains zero for 5 minutes, the device will automatically shut down |
| Environment For Storage & Use | Temperature 0-40°C Air humidity 30-75% |
| Expected Service Life | About 5 years |
| Length | 90.9 mm |
| Diameter | \varnothing 28 mm |
| Material | ABS + Stainless Steel |
| Warranty | 1 year |
| Certificates | ISO 13485, CE, ISO 10993 |
| OEM Service | Available |

| Program | Display | Output Frequency/Duration |
|-------------------|---------|--|
| Urge | A | 5Hz/3mins ,10Hz/5mins ,15Hz/5mins,20Hz/5mins,15Hz/5mins,10Hz/5mins ----- Total time=30mins |
| Stress | B | 10Hz/5mins.35Hz/5mins ,50Hz/10mins,35Hz/5mins,10Hz/5mins ----- Total time=30mins |
| Mixed | C | 10Hz/5mins, 20Hz/5mins ,30Hz/5mins, 40Hz/5mins ,50Hz/5mins ,40Hz/5mins ,30Hz/5mins ,20Hz/5mins, 10Hz/5mins -----Total time=45mins |
| After Care | D | Frequency Modulation Frequency is gradually increased from 2Hz to 10Hz in 4 seconds and then decreased back to 2Hz in another 4 seconds. Total cycle is 8 seconds. Pulse width is 200us. Total time=45 mins. |

Features and Benefits

- * Strengthens pelvic floor muscles
- * 4 pre-set modes
- * Large LCD user display
- * Load detection
- * Automatic power off
- * Compact and Lightweight
- * Safe and Easy to use
- * Recommended by Professionals
- * Value for money

OPERATIONS **i) Place the probe** - Connect the probe to the lead wire. - Insert the vaginal probe and / or the anal probe. - Connect the lead wire to machine. **ii) Turn the device on /off** Press power KEY . **iii) Adjust the Intensity** You can increase intensity by pressing the INTENSITY UP KEY and decrease by pressing INTENSITY DOWN KEY . You can see the adjustment by referring to the numbers of LCD level bars. **iv) Select a program** Use MODE KEY to choose a desirable program. After pressing the MODE KEY , it will go to the next mode and start with zero intensity. **v) After Use** Turn the unit off. Remove the probe and clean it.



Real picture display

Accessories

* vaginal probe * Instruction manual * 1x Leadwire * Neck Cord * 1x Lubricant gel sachet * Soft pouch * 2 x AAA batteries

SAFETY INFORMATION

1. Inspect the equipment before using it. 2. It is advised to get the advice from medically trained personnel before using the device. 2. The treatment is a personal ordination, so DO NOT lend probe to anyone else 3. People with implanted electronic equipment such as pacemakers and intracardiac defibrillators must not be treated with the device. 4. Do not use the device over the abdomen during pregnancy. 5. If high frequency surgical equipment is connected, stimulation should not take place. 6. Do not use the device in the vicinity of shortwave or microwave therapy equipment as this may affect the output power of the unit. 7. Keep the device out of reach of children. 8. If tissue irritation should occur, discontinued the treatment temporarily. If problems continue, contact your doctor. Hypersensitivity can occur in isolated cases. The problem usually disappears when probes or gel are changed to another type. 9. Never open the battery cover during stimulation in order to avoid electrical shock. 10. Turn off the device before removing the probe. 12. Use this device only with the leads and accessories recommended by the manufacturer

FAQ

Q:Do you accept OEM orders? A:Yes, you can place OEM and ODM order. In fact most of our orders are OEM orders with customised brand.And we are proudly the supplier of many famous brand in the industry. **Q:Does your product conform to our standard?** A:As a serious supplier of medical products, we strictly apply the European and USA regulations including ISO13485CE and 510K. Which covers the requirements of most of the countries. If your country still needs a special certificate, please contact us, we conform to some special standards too. **Q:How can I test the quality?** A:You are welcome to get samples from us and test them. All of our customers are very satisfied with the quality, they make comments that according to comparing test our quality is superior and with our products they have a very low rate of complaints. **Q:What's your payment terms and methods?** A:We accept only 100% payment before shipment. Usually by T/T banking transfer. Western Union, Moneygram, PayPal are all accepted. For trial order and sample order, Trade Assurance payment is preferred, FYI you can pay by T/T or credit card for Trade Assurance. **Q:How do you ship the goods?** A:We are very skilled in arranging the transport by sea, by air and by courier express. As we ship a lot we have a good discount rate for you. Also we can ship to your forwarder if you indicate one.



Guangzhou Janejoy Medical Technology Co., Ltd

+8613535187404

JaneJoy@therapy.org.cn

medicalcare-products.com

Building A2, Loteam Creative Park, No. 251, Kehua Street, Tianhe District, Guangzhou, China